


store your N95

What you need

View the PPE videos at med.emory.edu/PPE

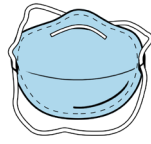

N95 Respirator



Paper storage bag with handles

Removal

1 **Sanitize** gloves.

2 **Pinch** bottom strap and **pull** far over head.
Do not let straps touch your face.

3 **Pinch** top strap and **pull** far over head.
Do not let straps touch your face as you **remove** the N95.

4 **Position** and hold N95 facedown.

Storage

5 **Place** storage bag on clean, flat surface.

6 **Place** N95 facedown into the storage bag.
Avoid touching inside or outside of bag.

7 **Sanitize** gloves.

Retrieval

8 **Sanitize** gloves.

9 **Remove** N95 from the storage bag.
Avoid touching inside or outside of bag.

10 **Put on** N95, ensuring proper seal.
Ensure straps are not crossing.

11 **Sanitize** gloves.